



**Step 1:** Use the provided measuring spoon and test tube to mix a solution of 10 mg of substance with 5 ml of water.

**Step 2:** Dip the strip up to the “MAX” line into your solution for 60 seconds.


**Step 3:** Allow the strip to dry for 1 - 3 minutes.

**Step 4:** Compare your results.


**1 Line = Fentanyl Detected** ☠️



**2 Lines = NO Fentanyl**

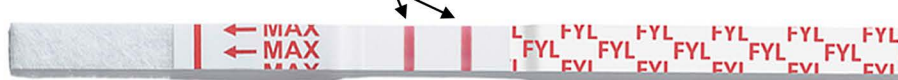

**2 Faint Line = NO Fentanyl**

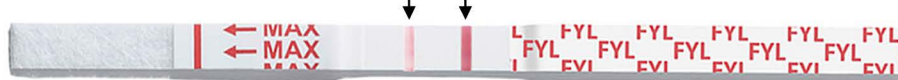

## Safety & Liability

A negative test result does NOT guarantee safety. Fentanyl can be unevenly distributed throughout a sample. One half of your substance may be completely free of fentanyl while the untested half may contain a lethal dose.

The use of this product constitutes agreement with TN Scientific's terms and conditions.

**Video Instructions**

