

FENTANYL TEST STRIP INSTRUCTIONS

(WITHOUT MEASURING EQUIPMENT)

STEP 1: ADD WATER UP TO THE '5 mL WATER FILL LINE' AS INDICATED ON THE PACKAGE, AND MIX IN A 10 MG SAMPLE.

***APPROXIMATELY THE SIZE OF HALF A GRAIN OF RICE.**




STEP 2: DIP THE STRIP TO THE "MAX" LINE INTO YOUR SOLUTION FOR 60 SECONDS.



STEP 3: LET THE STRIP DRY 1-3 MINUTES, THEN CHECK RESULTS.

***FAINT LINE IS A LINE.**

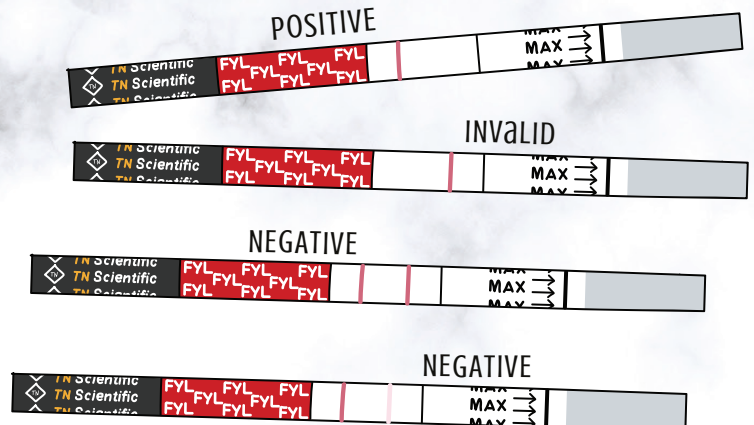

(C) (T)