

XYLAZINE / FENTANYL TEST STRIP INSTRUCTIONS


(WITHOUT MEASURING EQUIPMENT)

STEP 1: ADD WATER UP TO THE '5 mL WATER FILL LINE' AS INDICATED ON THE PACKAGE, AND MIX IN A 10 MG SAMPLE.

***APPROXIMATELY THE SIZE OF HALF A GRAIN OF RICE.**



STEP 2: DIP THE STRIP TO THE "MAX" LINE INTO YOUR MIXTURE FOR 60 SECONDS.

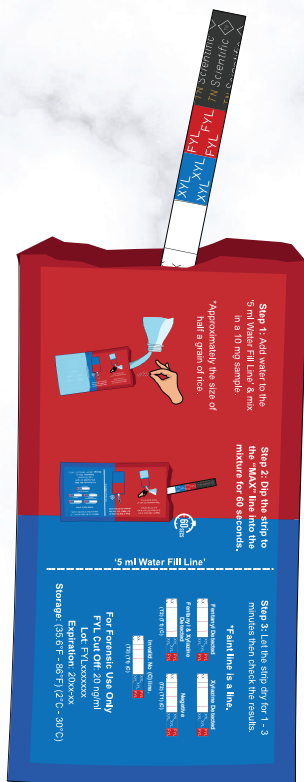

STEP 3: LET THE STRIP DRY 1-3 minutes, then check results.

***FAINT LINE IS A LINE.**

